Twin Bridges Business Park
222 Salford Road
Salford, Greater Manchester M6 6PU, U.K.
Tel: +44(0)20 8691 1414 Fax: +44(0)20 8649 8570
E-mail: sales@sle.co.uk Website: www.sle.co.uk

COLLODION ADHESIVE

Data Sheet

SLE PRODUCT CODE B0005/10
Tubes of Colloidion Adhesive for EEGEP Stick-on electrodes. 10 Tubes in an airtight container.

SLE PRODUCT CODE B0005
Tubes of Colloidion Adhesive for EEGEP Stick-on electrodes. 20 Tubes in an airtight container.

SLE PRODUCT CODE B0005/ALV020
Tubes of Colloidion Adhesive for EEGEP Stick-on electrodes. 20 Tubes in an airtight container.

SLE PRODUCT CODE B0005/50
Tubes of Colloidion Adhesive for EEGEP Stick-on electrodes. 50 Tubes in an airtight container.

INDICATIONS

- 1) Use the applicator to hold the electrode in place.
- 2) Apply the SLE Colloidion Adhesive.
- 3) Press the button on the applicator to dry the Colloidion with air from the compressor.
- 4) Cut through the hole in the centre of the electrode via a syringe with an blunt needle (cautiously).

INGREDIENTS

Colloidion for permeable membranes
Diethylene Glycol

FLASH POINT

-45°C (minus forty five degrees Centigrade)
EXTINGUISHING MEDIA Water or Vaporising Liquids.
This product is immiscible in water

FIRST AID

- EYES** Irrigate thoroughly with water for at least 10 minutes. Obtain medical attention.
- LUNGS** Remove from exposure, rest and keep warm. In severe cases, or if exposure has been great, obtain medical attention.
- MOUTH** Wash out mouth thoroughly and give water to drink.
- SPILLAGE** Absorb on an inert absorbent, transfer to container and arrange removal by disposal company. Wash site of spillage thoroughly.

CONSIGNMENT BY AIRFREIGHT

Consignment must be subject to strict packing & certification rules for Hazardous Goods. This is charged extra.

CAUTIONS

This product is **Extremely Flammable**, use only in well ventilated areas, and away from sources of ignition.

May cause skin irritation.

Not to be used as a general adhesive.

Dispose of unused product in accordance with in house practice.

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